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Beskrivelse

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Freehand Tray AA5810

Description

This is a quality white melamine tray which has been ergonomically designed with a plastic handle that clips onto the hand holes in the tray to give a large carrying handle. This allows the tray to be carried one-handed and be well balanced.

It is provided with a Dycem non-slip mat, which prevents things sliding around, even when the tray is at an angle. The tray is large enough for a full meal, and has deep sides.

- Size 445 x 330mm (17 1/2 x 13")

Assembly

- 1) Place the tray on a firm worktop or table.
- 2) Hook one end of the handle under one of the tray's finger holds.
- 3) Spring the handle end up into position to

firmly grip the tray (an audible click will ensure correct locking).

- 4) Spring the other end of the handle into position within its fingerhold and clip up to lock.

To release the handle, steady the tray and press one end of a handle clip down. This will spring the handle off the locking position. Repeat at other end of the handle.

Dycem Tray Liner

The liner should be periodically washed in a mild detergent. It should not be used wet as this will reduce its non-slip qualities.

Cleaning & Care

Wipe the tray over with a solution of washing up liquid and water, dry immediately.

MADE IN THE UK
MANUFACTURED FOR HOMECRAFT ABILITYONE

Homecraft AbilityOne

Homecraft AbilityOne

Shelley Close, Lowmoor Business Park,
Kirkby-in-Ashfield,
Nottinghamshire, NG17 7ET, UK
Telephone: +44 (0)1623 754047 Facsimile: +44 (0)1623 755585

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